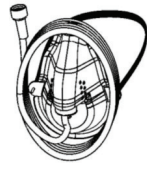
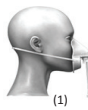
CAPNOGRAPHY - OXYGEN MASKS - PEDIATRIC

"A Consistent Means to Monitor Breathing"

REF CO2-04; CO2-04-SLM

MD   Rx Only

UK CA 8532
Emargo Consulting (UK) Limited
c/o Cr 360—UL International
Compass House, Vision Park Histon
Cambridge, CB24 9BZ
United Kingdom

EU REP

EMERGO EUROPE
Westervoortseindijk 60
6827 AT Arnhem
The Netherlands

CH REP

MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland

MADE IN CANADA

IFU0008Q
Issue Date: 2026-02-23

50 Alliance Blvd. Barrie, Ontario
Canada L4M 5K3
705-726-9383
705-728-9537 FAX
www.southmedic.com

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

EN INTENDED USE: The CAPNOXYGEN® Mask is a medium concentration single use mask for delivering supplemental oxygen and monitoring breathing by providing a means to sample exhaled carbon dioxide. Standard connectors for the oxygen tubing and a standard female Luer connector for the gas sample line are provided. The mask is intended to be used for monitoring non-intubated patients who are breathing spontaneously.

EXPECTED CLINICAL BENEFITS: The CAPNOXYGEN® Mask enables the delivery of supplemental oxygen up to 15+ LPM, targeting an oxygen saturation range in the treatment of hypoxemia and relief of shortness of breath. The device enables continuous EtCO₂ monitoring for nasal and oral breathing which can provide an early warning of potential respiratory conditions.

WARNINGS: The device facilitates EtCO₂ sampling when used in conjunction with capnography equipment. The displayed CO₂ values or any other parameter/index driven from CO₂ will vary with the sampling port position, oxygen flow rate and respiratory pattern.

Oxygen Flowrate should be maintained at 5 L/min or more in order to avoid rebreathing exhaled carbon dioxide that can build up inside the mask (1).

CAUTIONS: Oxygen in the presence of an ignitable material and an ignition source can cause a fire resulting in severe personal injury or death. Do not smoke while using this device. Single use product not to be reused due to effect on performance, cross contamination and risk of infection. The product is disposable and is not intended for use greater than 29 days. The device has a 5-year shelf life from the date of manufacture. Date of manufacture is stated on labelling. Store device in original packaging until use. Inspect product and packaging prior to use. Do not use if product appears to be damaged.

REPORTING SUSPECTED SIDE EFFECTS: If you suspect that use of this medical device is associated with an adverse occurrence please report the details of the incident to your health authority, and your local distributor or directly to the manufacturer at www.southmedic.com.

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