Figure 1

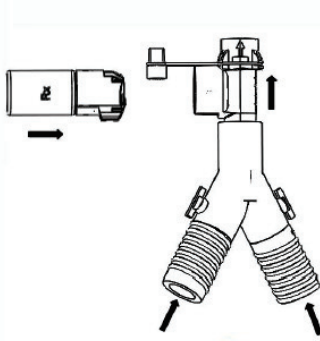
Figure 2

REF SMDIA-1300

**Ventilator Adapter**  
for use with MDI  
15MM OD X 15MM Flow OD to ID

Not made with natural rubber latex

50 Alliance Blvd., Barrie, ON, Canada L4M 5K3  
705-726-9383  
705-728-9537 FAX  
www.southmedic.com  
Made in Canada

IFU02651

## Ventilator Adapter for use with MDI with or without dose counter. Accepts metal or plastic tipped standard MDI canisters.

The Southmedic MDI Adapter is indicated for ventilated patients requiring bronchial medication.

**Intended Use:**  
The SMDIA-1300 MDI Adapter is intended to be used with an MDI for intermittent delivery of prescribed aerosol medications dispensed by MDIs. It is intended to be used only when connected to ventilator tubing or tracheal tubes.

- Directions for Use:**
1. Connect the SMDIA-1300 Adapter in line.
  2. Ensure that the air flow arrows indicated on the adaptor are pointing in the direction of flow.
  3. Remove the plug on the tether.
  4. (a) When using an MDI with counter, refer to Figure 1.
  5. Administer medication from the MDI canister as prescribed.
  6. When complete, replace plug.
  7. Dispose of in the same manner as the circuit in which the MDI Adapter is used.

**Warnings:**  
Single patient use not to be left in circuit for more than 29 days, due to effect on performance, cross-contamination and risk of infection. Oxygen in the presence of an ignitable material and an ignition source can cause a fire, flash fire, explosion, or injury and death. Do not smoke while using this device.

**Contraindications:**  
Patients who cannot tolerate the addition of 16cc of dead space.

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