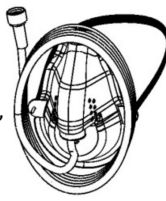
OPHTHALMIC

CAPNOGRAPHY - OXYGEN MASKS - "A Consistent Means to Monitor Breathing"

REF CO2-03; CO2-03-SLM; CO2-03-SLM-14

Intended Use: The CAPNOXYGEN® Mask is a medium concentration single use mask for delivering supplemental oxygen and monitoring breathing by providing a means to sample exhaled carbon dioxide.

Warning: Hypercapnia will not be reliably detected by sampling through this device, the displayed CO2 values or any other parameter/index driven from CO2, will vary with the sampling port position, oxygen flow rate and respiratory pattern.

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Caution: Oxygen in the presence of an ignitable material and an ignition source can cause a fire resulting in severe personal injury or death. Do not smoke while using this device.

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EMERGO EUROPE Westervoortsejdijk 60 6827 AT Arnhem The Netherlands

MedEnvoy Switzerland Gotthardstrasse 28 6302 Zug Switzerland

UK CA0086 CE 2797 Emergo Consulting (UK) Limited c/o Cr 360-UL International Compass House, Vision Park Histon Cambridge, CB24 9BZ United Kingdom

50 Alliance Blvd. Barrie, Ontario Canada L4M 5K3 705-726-9383 705-728-9537 FAX thebetteroxygenmask.com

EMERGO EUROPE Westervoortsejdijk 60 6827 AT Arnhem The Netherlands

MedEnvoy Switzerland Gotthardstrasse 28 6302 Zug Switzerland

UK CA0086 CE 2797 Emergo Consulting (UK) Limited c/o Cr 360-UL International Compass House, Vision Park Histon Cambridge, CB24 9BZ United Kingdom

WARNING

Oxygen Flowrate should be maintained at 5 L/min or more in order to avoid rebreathing exhaled carbon dioxide that can build up inside the mask.

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