REF

Oxygen Tube Length	7 feet		14 feet	
Sampling Line	SLF 8"	SLM 8'	SLF 8"	SLM 15'
OxyMask™ EtCO <sub>2</sub> Kid	OK-2125-8	OK-2125-8SLM	-	-
OxyMask™ EtCO <sub>2</sub> Adult	OM-2125-8	OM-2125-8SLM	OM-2125-14	OM-2125-14SLM
OxyMask™ EtCO <sub>2</sub> Plus	OP-2125-8	OP-2125-8SLM	OP-2125-14	-

**Intended Use:** A minimal contact, open oxygen delivery device for use with USP medical grade oxygen for monitoring breathing by providing a means to sample exhaled CO<sub>2</sub>.

**Warning:** Hypercapnia will not be reliably detected by sampling through this device, the displayed CO<sub>2</sub> values or any other parameter/index driven from CO<sub>2</sub> will vary with the sampling port position, oxygen flow rate and respiratory pattern.

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**Caution:** Oxygen in the presence of an ignitable material and an ignition source can cause a fire resulting in severe personal injury or death. Do not smoke while using this device. Single use product not to be reused due to effect on performance, cross contamination and risk of infection.

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EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

MedEnvoy Switzerland  
Gotthardstrasse 28  
6302 Zug  
Switzerland

UK  
CA 0086  
CE 2797  
Emergo Consulting (UK) Limited  
c/o Cr 360—UL International  
Compass House, Vision Park Histon  
Cambridge, CB24 9BZ  
United Kingdom

50 Alliance Blvd.  
Barrie, Ontario  
Canada L4M 5K3  
705-726-9383  
705-728-9537 FAX  
thebetteroxygenmask.com

EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

MedEnvoy Switzerland  
Gotthardstrasse 28  
6302 Zug  
Switzerland

UK  
CA 0086  
CE 2797  
Emergo Consulting (UK) Limited  
c/o Cr 360—UL International  
Compass House, Vision Park Histon  
Cambridge, CB24 9BZ  
United Kingdom

Oxygen Flow Settings	Approximate O <sub>2</sub> Concentrations
1 LPM	24% - 25%
2 LPM	27% - 32%
4 LPM	34% - 40%
6 LPM	42% - 48%
8 LPM	49% - 55%
10 LPM	53% - 58%
12-15 LPM	59% - 65%

\*Actual FIO<sub>2</sub> will vary. Breath rate, depth of breathing and distance of the diffuser from the face will affect performance.  
\*\* Actual flow rate required should be prescribed in conjunction with pulse oximetry.

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